Instructions for use Twain / WIA Software für DIAGNOcam 2170

Always be on the safe side.

#### Manufacturer:

Kaltenbach & Voigt GmbH Bismarckring 39 D-88400 Biberach www.kavo.com

# Distributed by:

KaVo Dental GmbH Bismarckring 39 D-88400 Biberach Phone +49 7351 56-0 Fax +49 7351 56-1488 Table of contents

# Table of contents

| 1 | Desci | iption of the Product                                     | . 2 |
|---|-------|---|-----|
|   | 1.1   | Application area  | 2   |
| 2 | Comr  | nissioning  | 3   |
|   | 2.1   | Installation with XP, WIN 7 32/64 bit WIN 8 32/64 bit     | 3   |
|   | 2.2   | Driver installation of the USB user interface only for XP | 7   |
| 3 | Opera | ation   | 10  |
|   | 3.1   | Description user interface                                | 10  |
|   | 3.2   | Changing the window size                                  | 10  |
|   | 3.3   | Creating images   | 10  |
|   | 3.4   | Comparing images  | 11  |
|   | 3.5   | Transmit to external X-ray software                       | 11  |

1 Description of the Product | 1.1 Application area

# 1 Description of the Product

# 1.1 Application area

With the Twain / WIA software, the user can transmit DIAGNOcam images to other software such as to X-ray software or to image processing programs that support Twain or WIA.

#### Note

From today's perspective, we recommend using Twain as the application software or X-ray software run on 32 bit. WIA supports future 64 bit applications. WIA cannot be used on computers with XP operating systems.

## 2 Commissioning

**Requirement** Carry out installation as administrator!

2.1 Installation with XP, WIN 7 32/64 bit WIN 8 32/64 bit

This chapter describes the procedure following the insertion of the CD.

 Double click the file, DIAGNOcam.... .exe in the root directory of the CD. This starts the installation.

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• Confirm the user account control with "Yes", XP does not provide this window.

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Select language and confirm with "OK".

Confirm setup by clicking "Continue".

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It is recommended to select the proposed folder and to proceed by clicking "continue".

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Select installation type according to the requirements of the imaging software, then proceed with "continue".

#### The installation procedure starts.

| $\sim$ |   |  |
|--------|---|--|
|        | • |  |
|        |   |  |
|        | ш |  |
|        |   |  |

### Note

The following window does not appear with XP, another window is displayed, confirm this by clicking "Continue installation".

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The driver is installed by clicking "Install"

### Note

The following window does not appear with XP, another window is displayed, confirm this by clicking "Continue installation".

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• Software is installed with continue "Install", the installation procedure starts.

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Select "Restart later"

2 Commissioning | 2.2 Driver installation of the USB user interface only for XP

It is recommended to restart the computer.
Select "Yes, restart computer now" and click "Finish".

2.2 Driver installation of the USB user interface only for XP

### Note

Every USB user interface must be installed on the PC / laptop the first time with XP only. This is no longer necessary with WIN 7 or WIN 8.

2 Commissioning | 2.2 Driver installation of the USB user interface only for XP

From the DIAGNOcam, connect the USB cable with the PC / laptop, the following window opens:

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 Select assistant for hardware search "No, not this time" and proceed with "continue".

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2 Commissioning | 2.2 Driver installation of the USB user interface only for XP

Select "Automatically install software" and proceed with "continue".

Select "Continue installation" and continue.

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• Complete installation by clicking "Finish".

3 Operation | 3.1 Description user interface

# 3 Operation

## 3.1 Description user interface

- Plug in DIAGNOcam.
- Start the external software.
- Start from this SW Twain application (procedure according to the relevant SW).

### Note

Please use WIA if the software does not support TWAIN

Window 1 top left: Current live image Window 2 top right: Saved image Window 3 bottom: Selection bar for saved images

# 3.2 Changing the window size

- The window size can be changed by clicking and shifting the frame.
- This can be reversed by clicking the symbols "Reset layout" in the top bar of the screen.

# 3.3 Creating images

- Images can be saved by clicking the symbol "capture" or by a mouse click in window 1 or by activating the ring switch on DIAGNOcam.
- These are displayed in window 2 and in window 3 in the 1. position on the left. Further images are displayed in window 3 in the 1. position on the left whereby the previous images are shifted one position to the right.

3 Operation | 3.4 Comparing images

## Note

If the ring switch or capture is activated in the clipboard, only black images are saved.

## 3.4 Comparing images

Saved images in window 3 are displayed by clicking in window 2 and can be compared with the live image.

#### 3.5 Transmit to external X-ray software

Mark the images to be transmitted in window 3 by clicking (images will acquire blue frames) and transmit to external software by clicking the symbol "Save selected images".

#### Note

- Impact of the application or X-ray software on the image transmission
- 1. Although several images have been selected, only one can be transmitted.
- 2. The image selection sequence does not match with the image transmission sequence.
- 3. Availability of the images following the image transmission.
- 4. Twain/WIA does not close automatically after the image transmission.

Symbol "Capture", recording of images

Symbol "Select all", all images in window 3 are marked

Symbol "Save selected images", marked images are transmitted to the external software (X-ray software).

Symbol "Disable ring switch", ring switch is activated/deactivated by DIAGNOcam. 3 Operation | 3.5 Transmit to external X-ray software

#### Symbol "Camera options", selection of the control panel camera options

| /ideoProcAmp       |         |          |              |   |
|--------------------|---------|----------|--------------|---|
| Camera Controls:   |         |          | Auto         | - |
| Prinkhusse         |         | 128      |              |   |
| Contrast           |         |          | - <b>-</b> - |   |
| AGC:               | 100     | 0        | ~            |   |
| AEW:               |         | 140      |              |   |
| AEB:               |         | 130      | - F          |   |
| AEC                |         | <i></i>  |              |   |
| 🔽 AEC dig. Averagi | ng      | 🔽 Flip H | orizontal    |   |
| ✓ Sharpness        |         | 🔽 Flip V | ertical      |   |
| default            | DX Vers | ion:     |              |   |
|                    |         |          |              |   |

If the teeth are displayed too brightly, we recommend changing the AEW values accordingly.

By clicking "default", the values are reset to the factory setting.

Symbol "Delete images on close", on activation, all images are deleted after the closing of TWAIN/WIA.

Symbol "Reset layout", resetting of changed window sizes.

Symbol "Delete selected images", marked images are deleted.

Symbol "Close", Twain/WIA is closed.

1.010.4080 · kf · 20130625 - 01 · en